

# Shiley®

## Pediatric Tracheostomy Tube

Product Designation: PT



### Instructions for use

THESE INSTRUCTIONS ARE APPLICABLE ONLY TO THE PEDIATRIC TRACHEOSTOMY TUBE AND SHOULD BE READ CAREFULLY BEFORE USING THE DEVICE.

PLEASE POST FOR USE BY ALL TRACHEOSTOMY CARE PERSONNEL.

#### Contents of Package:

This package contains a Shiley Pediatric Tracheostomy Tube, an obturator and a neck strap.

#### CAUTION: Single Patient Use Only.

**WARNING: Ethylene oxide sterilized. Sterile only if protective package is not opened, damaged or broken. Do not resterilize. Do not expose to temperatures above 120°F.**

Product Designation	I.D. mm	O.D. mm	Length mm
00 PT	3.1	4.5	39
0 PT	3.4	5.0	40
1 PT	3.7	5.5	41
2 PT	4.1	6.0	42
3 PT	4.8	7.0	44
4 PT	5.5	8.0	46

**Note:** I.D. is the inside diameter of the cannula. O.D. is the outside diameter of the cannula. Length is the distance from the neck plate to the distal tip of the tube.

### Indications for Use:

1. To provide the maximum airway, the largest size tube which will conform to the patient's anatomy should be chosen.
2. Expose the anterior tracheal wall using the standard infant tracheostomy technique.
3. Insert the obturator into the tube. The obturator must be fully seated before the tube is inserted into the trachea. A thin film of water soluble lubricant should be applied to the tube and protruding portion of the obturator to facilitate ease of inserting the tube.
4. After the tube is properly in place, remove the obturator.
5. Attach the neck strap to the tube. Flex the infant's head forward and tie the neck strap. When properly adjusted, you should be able to place one finger between the neck strap and the infant's neck.
6. The integral 15 mm connector on the proximal end of the tube fits standard ventilator equipment.

**CAUTION: Do not use any solutions other than sterile saline to clean any part of this tube.**

### Rx ONLY

Tyco Healthcare Group LP  
Nellcor Puritan Bennett Division  
Pleasanton CA USA  
1-800-NELLCOR

**tyco**  
Healthcare

Made in Mexico  
© 2003 Nellcor Puritan Bennett, Inc.  
All rights reserved.