# I N S T R U C T I O N S F O R U S E

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**DURA·NEB<sup>®</sup>**  
**3000**  
PORTABLE AEROSOL SYSTEM

## QUICK GUIDE, GETTING STARTED

PARI has designed this Quick Guide as a ready reference to keep you on the go.

- 1 Read the "Instructions For Use" manual thoroughly before operating unit.
- 2 If using the battery, make sure the battery is fully charged by checking the battery capacity indicator located on the front of the DURA-NEB 3000.

BRIGHT GREEN	Battery fully charged.
YELLOW	Approx. 1/3 battery capacity remaining.
RED	Recharging is a must (Only 1-2 treatment are left).

If using a wall outlet or 12V vehicle outlet, make sure all cords are connected properly

- 3 Pour medication into the nebulizer cup according to your prescription.
- 4 Assemble the nebulizer as described in the Instructions For Use.
- 5 Attach optional mask as instructed by your physician.
- 6 Attach the nebulizer tubing to the air outlet connector on the front of the DURA-NEB 3000.
- 7 Turn on the compressor.
- 8 Breathe in the medication mist through the mouthpiece or mask as instructed by your physician.
- 9 After your treatment is completed, turn compressor off and disconnect the nebulizer tubing from the air outlet connector. Clean and/or disinfect the nebulizer cup as shown in the Instructions For Use.

TEAR OFF AND FOLD QUICK GUIDE, THEN PLACE INTO CARRYING CASE FOR REFERENCE.

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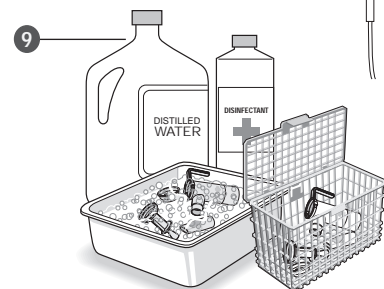
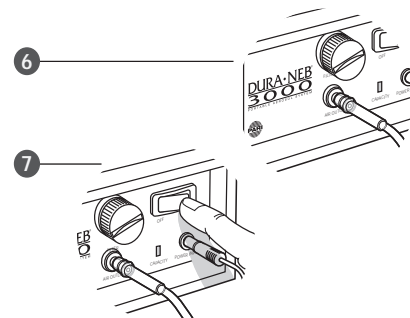
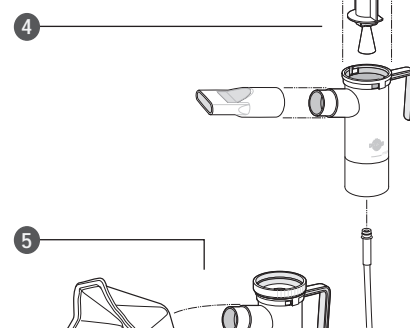
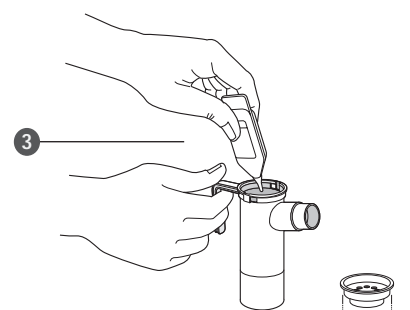
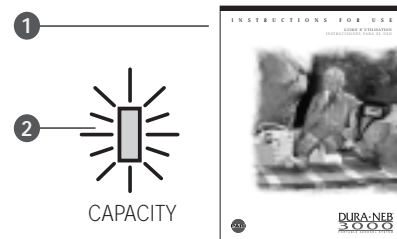
## QUICK GUIDE, BATTERY RECHARGING AND REMOVAL

The DURA-NEB 3000 battery may be recharged using any wall **A** or 12V DC **B** vehicle outlet.

- Make sure the compressor is turned off.
- Plug the small round end of the charger/adaptor (AC or DC) into the socket on the front panel of the DURA-NEB 3000 marked "POWER INPUT".
- Plug the other end of the charger/adaptor into the wall or vehicle outlet. If using a vehicle outlet, the engine must be running in order to recharge the battery.
- To ensure the battery receives a complete charge, recharge unit overnight. Recharging may take up to 8 hours, depending on the current charge level of the battery. The unit has been designed to protect the battery from overcharging.

### BATTERY REMOVAL

- 1 Turn the unit face down on a flat surface. Gently lift up the loose end of the grey trim to expose the battery access door. Insert a flat screwdriver, coin, spoon handle, or flat object into the door notch.
- 2 Gently push in and LIFT screwdriver UP to open battery door. Remove door by lifting up.
- 3 Tap unit against your palm at door opening until battery is dislodged into your hand.

### BATTERY REPLACEMENT

- 4 Turn unit face up. Insert the battery prongs first, as illustrated. Replace battery door, then press grey trim back into place.

RECHARGE NEW BATTERY BEFORE FIRST USE.

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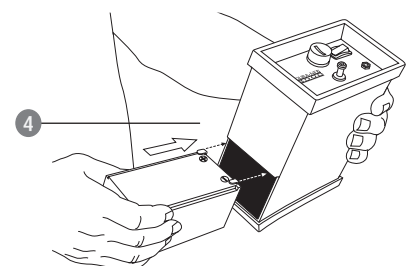
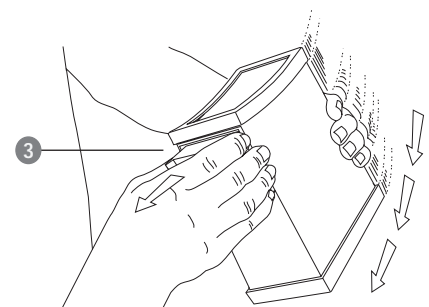
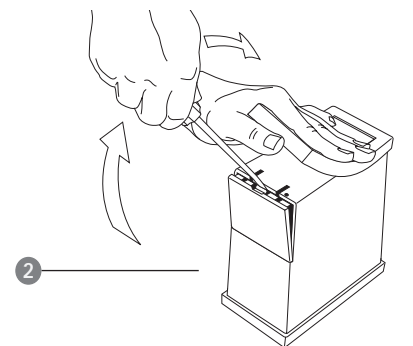
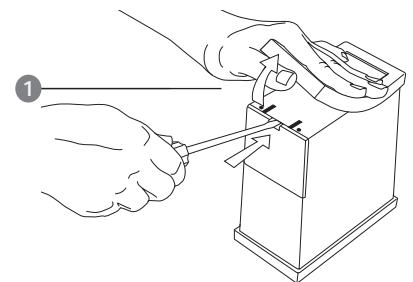
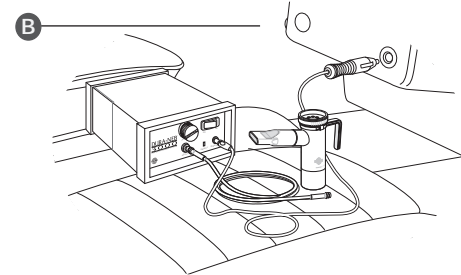
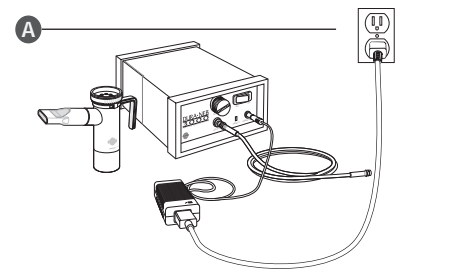
Table Of Contents:

- I. Introduction ..... 1
- II. Features And Functions Of The DURA-NEB 3000 ..... 2
- III. Specifications ..... 3
- IV. Getting Started ..... 4
- V. Safety Instructions ..... 5-6
- VI. Set-Up ..... 7
- VII. Internal Battery Care (for 20F30-LC+ only) ..... 9
- VIII. Proper Care And Maintenance ..... 11
- IX. Procedure For Service ..... 13
- X. Limited Warranty ..... 13-14
- XI. Important Warranty Information ..... 14

I. Introduction

These Instructions For Use contain information on the DURA-NEB 3000 Portable Aerosol System (DURA-NEB 3000). Separate Instructions For Use are included with the enclosed PARI nebulizer.

Before using your DURA-NEB 3000, you should read and understand the entire Instructions For Use for both the DURA-NEB 3000 and PARI nebulizer. Make special note of the internal battery charging procedure on Section VII and all danger, warning and caution statements. Save this guide for future reference.

II. Features And Functions Of The DURA-NEB 3000

- ① DURA-NEB 3000 Portable Aerosol System
- ② Battery Access Door
- ③ On-Off Switch
- ④ Filter
- ⑤ Air outlet
- ⑥ Power Input
- ⑦ Battery Capacity Indicator
- ⑧ AC Multi-Voltage Charger/Adapter For Worldwide Use
- ⑨ DC Charger/Adapter (cigarette lighter or vehicle outlet use)

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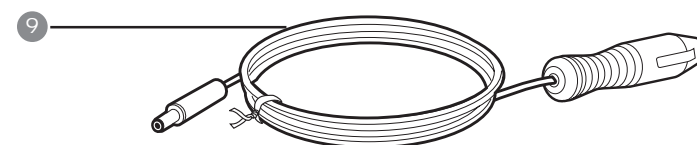
### III. Specifications

**COMPRESSOR:**

Weight (with battery): .....	5.0 lbs. (2.3 Kg)
Weight (without battery): .....	2.4 lbs. (1.1 Kg)
Dimensions: .....	3 7/16" x 6 1/8" x 6 7/8" (8.7 x 15.6 x 17.5 cm)
Ambient Temperature Range: .....	50° F - 104° F, (10° C - 40° C)
Maximum Pressure: .....	25 ± 5 psi

**AC/DC CHARGER/ADAPTER:**

Input Voltage Range: .....	90V AC to 265V AC
Input Frequency: .....	47-63Hz
DC Output Voltage: .....	15V DC
Rated Output Current: .....	2.0A
Wattage: .....	30W
Insulation: .....	Grounded

**INTERNAL BATTERY:**

No. of Treatments Per Charge: .....	Up to 18 treatments when nebulizing 2.5 ml of medication using a PARI nebulizer
Charge Time: .....	8 Hrs. (initially and when depleted)

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### IV. Getting started

1. Check the DURA-NEB 3000 packing box and make sure all components listed below are included.

**DESCRIPTION:**

	<b>PART NO.</b>
DURA-NEB 3000 Portable Aerosol System (With Battery) .....	20F30-LC+
DURA-NEB 3000 Portable Aerosol System (Without Battery) .....	20F31-LC+
Deluxe Carrying Case .....	20F2999
Multi-Voltage AC Charger/Adapter .....	20F6000
DC Charger/Adapter (cigarette lighter or vehicle outlet use) .....	20F2036
Air Filter (in compressor) .....	41F64
DURA-NEB 3000 Instructions For Use .....	20D0005
PARI Nebulizer Set .....	22F81

**OPTIONAL ACCESSORY:**

Internal Battery .....	20F1000
------------------------	---------

If you did not receive all of the components listed, please contact the authorized dealer where you purchased your DURA-NEB 3000.

2. Fill out warranty card completely and mail.
3. It is recommended that the customer purchase a back-up nebulizer in case of lost or damaged parts.
4. Contact PARI Patient Connection at 1.800.FAST.NEB (327.8632) and Press 2 if you have any questions about getting started.

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**▲ WARNING**

U.S. Federal Law restricts this device to sale by or on the order by a physician. Use only when prescribed by a physician and with the prescribed medication. Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury. This product contains dry natural rubber.

Use only with nebulizers and tubing authorized by PARI for this device. Other nebulizers and tubing may result in improper treatment.

**V. Safety Instructions**

As with all electrical and medical products, basic safety precautions should always be followed.

**▲ DANGER**

To reduce the risk of serious or fatal injury from electrocution:

1. Do not use or store product, especially the Charger/Adapter, where it can fall or be pulled into a bath tub, sink, water, or other liquid.
2. Do not reach for a product, especially the Charger/Adapter that has fallen into water or other liquid.

**▲ WARNING**

To reduce the risk of serious or fatal injuries from electrocution, fire or burns and to reduce the risk of damage to the unit:

1. Do not use the DURA-NEB 3000 in the presence of flammable anesthetics, in a hazardous atmosphere, or with flammable agents.

2. Do not operate with a damaged cord or plug. Do not operate if the unit has otherwise been damaged or exposed to any liquids inside the case.
3. Keep all electrical cords away from heated surfaces. Do not overload wall outlets or extension cords.
4. Never cover the vents of the AC Charger/Adapter during use. Blockage could cause excessive heat build-up. Never leave the Charger/Adapter in the DURA-NEB carrying case while the Charger/Adapter is in use.
5. Be sure the Charger/Adapter is unplugged prior to cleaning.
6. Do not open the case or disassemble the compressor unit (except for the Battery Access Door) or the Charger/Adapter. Refer all servicing to an authorized servicing center.

**▲ WARNING**

To reduce the risk of serious or fatal injury or improper treatment:

1. Use the product only for its intended use as described in the Instructions For Use. Do not use nebulizers or other attachments not recommended by the manufacturer for this device.

2. Close supervision is strongly recommended when this product is used by or near the physically or mentally impaired or children. Never use while sleepy or drowsy.
3. Never operate this product if it is not working properly or if any parts have been damaged. Return the product to an authorized service center for evaluation and repair.

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Conserve Estas Instrucciones

## VI. Set-Up

## CHOOSE THE APPROPRIATE POWER SOURCE

The DURA-NEB 3000 can be used with the following power sources:

- Rechargeable Internal Battery (optional)
- Multi-Voltage AC Charger/Adapter For Worldwide Use
- 12V DC Cigarette Lighter or Vehicle Outlet

**A TO USE THE DURA-NEB 3000 FROM THE WALL OUTLET (115V OR 230V)**

1. Connect the plug of the AC Charger/Adapter into the wall outlet **A-1**.
2. Plug the small round end of the AC Charger/Adapter into the socket on the front panel of the DURA-NEB 3000 marked "Power Input" **A-2**.
3. Read and follow the Instructions For Use included with your PARI nebulizer package.

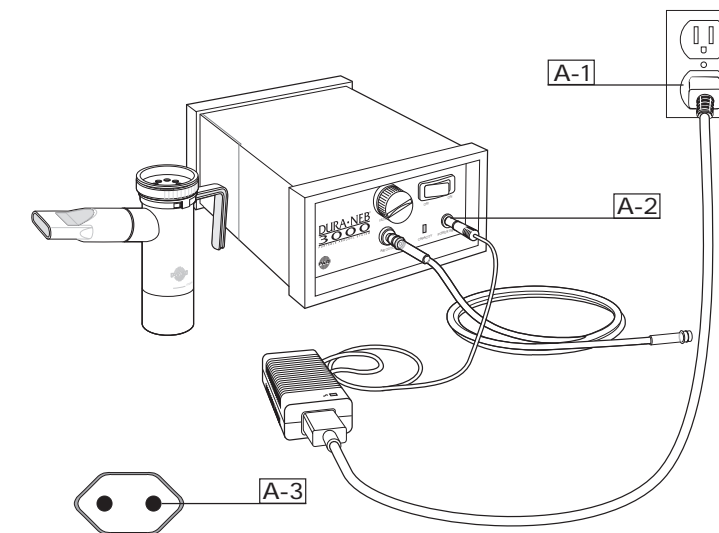
Note: The Multi-voltage Charger/Adapter comes equipped with an American Pattern Plug. You may need an Adapter Plug Set **A-3**, (sample plug shown) for most European Countries (Part No. 44F2207). The Charger/Adapter automatically converts the incoming voltage of the country visited. Manual switching is not required.

**B TO USE THE DURA-NEB 3000 FROM A 12V DC CIGARETTE LIGHTER OR VEHICLE OUTLET**

1. Plug the cigarette lighter end of the DC Charger/Adapter into the 12V DC cigarette lighter or vehicle outlet in the car, boat or motorhome **B-1**.
2. Plug the small round end of the DC Charger/Adapter into the socket on the front panel of the DURA-NEB 3000 marked "Power Input" **B-2**.
3. Read and follow the Instructions For Use included with your PARI nebulizer package.

**C TO USE THE DURA-NEB 3000 FROM THE INTERNAL BATTERY**

1. You must fully charge the internal battery before using for the first time. To charge the internal battery, follow the steps in Section VII.
2. As you take your treatment, please pay attention to the Battery Capacity Light so that the battery maintains a sufficient charge. The Battery Capacity Light will be green when the battery is fully charged and it will turn to yellow as the battery is depleted. When the Battery Capacity Light is red, the battery will need to be recharged (see Section VII). The battery will last longer if you recharge the unit as often as possible. Operating compressor when capacity light is red may damage battery. AVOID DRAINING THE BATTERY COMPLETELY, AS THIS WILL LOWER THE LIFE EXPECTANCY OF THE BATTERY.
3. Read and follow the Instructions For Use included with your PARI nebulizer package.

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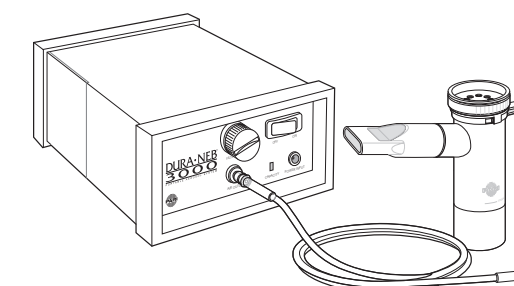
## C

## VII. Internal Battery Care (for 20F30-LC+ only)

Your battery is a gel, suspended cell battery, which does not require you to drain it completely before recharging. The battery will last the longest if you can recharge the unit after every treatment. The more you recharge the battery, the longer it will last. When the battery is new, the DURA-NEB 3000 will deliver up to 18 treatments when nebulizing 2.5 ml of medication using a PARI nebulizer. As time goes by, the battery capacity will diminish. The life of the battery is affected by how often you recharge it and how long you wait before recharging. The optimum care for the battery is to recharge it as often as possible.

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### TO RECHARGE THE INTERNAL BATTERY

The DURA-NEB 3000 battery may be recharged using any wall or 12V DC vehicle outlet. Make sure the compressor is turned off. Plug the small round end of the adapter into the socket on the front panel marked "Power Input". Plug the other end of the Charger/Adapter into the wall or vehicle outlet. If using a vehicle outlet, the vehicle must be running in order to recharge the battery. If using a wall outlet, make sure it is not being shared by another power consuming appliance. To ensure the battery receives a complete charge, recharge unit overnight. Recharging may take up to 8 hours, depending on the current charge level of the battery.

### D REMOVING THE INTERNAL BATTERY

1. Turn compressor face down on a flat surface.
2. Gently lift up loose end of grey trim to expose battery access door **D-1**.
3. Insert flat screwdriver, coin, spoon handle, or flat object into door notch.

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4. Gently push in and LIFT screwdriver UP to open battery door. Remove door by lifting up **D-2**.
5. Tap compressor against your palm at door opening until battery is dislodged into your hand. DO NOT pry battery out with screwdriver or any other object. If it is difficult to dislodge battery, tap compressor against hard surface until battery is dislodged into palm **D-3**.

### ▲ WARNING

Do not store medication, nebulizer, or other objects in battery compartment if no battery is installed.

### D REPLACING THE INTERNAL BATTERY

1. Turn compressor face up.
2. Insert battery, prongs first, as illustrated **D-4**.
3. Replace battery door.
4. Press grey trim back into place.
5. Charge new battery before first use (see section VII, To Recharge the Internal Battery).

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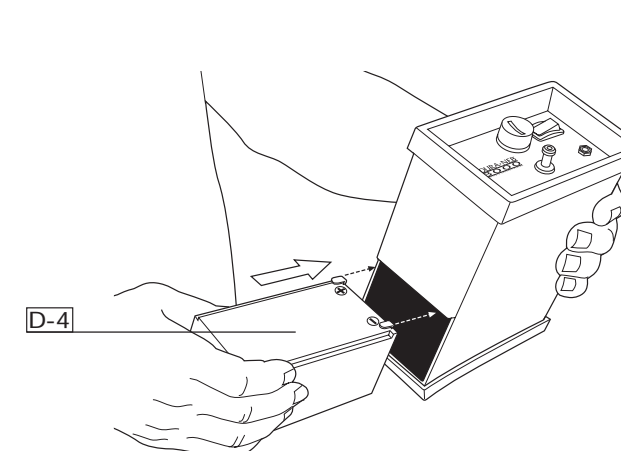
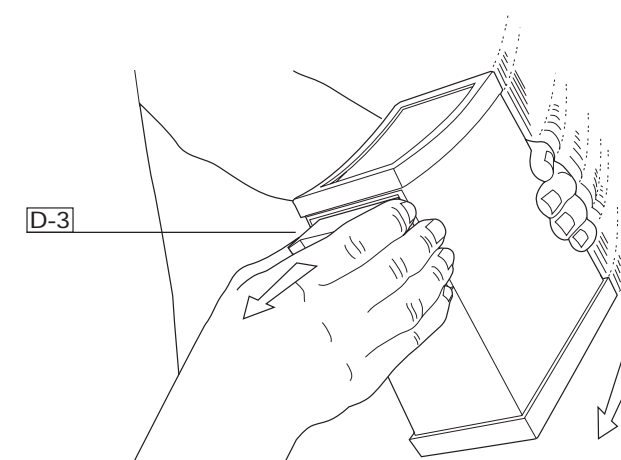
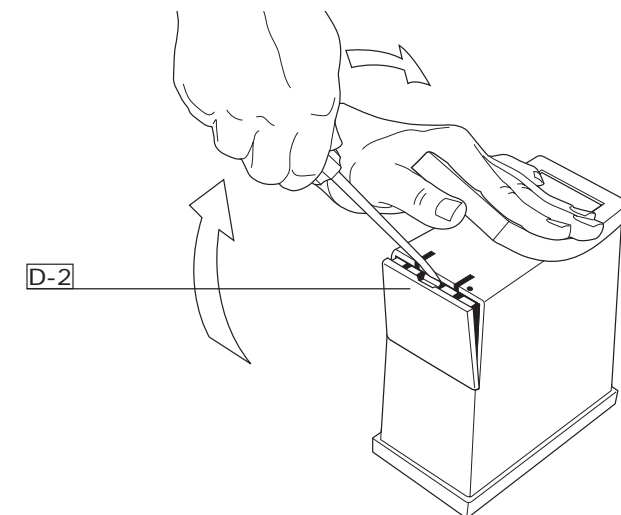
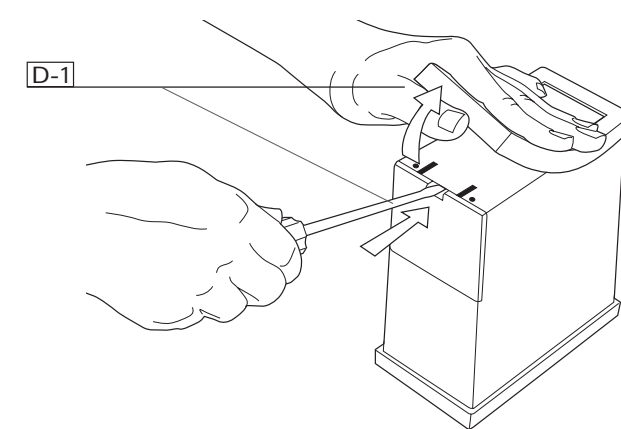
## VIII. Proper Care And Maintenance

### STORING THE DURA-NEB 3000

If you plan on storing the DURA-NEB 3000 for an extended period of time, recharge the internal battery overnight once every 2 months, unplug the compressor, then return it to storage. Under normal circumstances, as the battery nears the end of its life, it will require recharging more frequently. Contact your DURA-NEB 3000 dealer or PARI Patient Connection to purchase a replacement battery (part# 20F1000) when you notice diminished battery capacity. Changing the battery is strongly recommended when the battery is not operating properly. If the new battery does not correct the problem, then contact PARI Patient Connection for service, (see section IX).

Note: Excluding battery, there are no user serviceable parts inside the DURA-NEB 3000 compressor case. Service by a non-authorized DURA-NEB 3000 service center voids warranty.

### CLEANING THE DURA-NEB 3000 HOUSING

Use a damp cloth to wipe the exterior of the DURA-NEB 3000 housing. Do not use any commercial cleaners on the unit.

### MAINTENANCE

Never operate the DURA-NEB 3000 if it is damaged or improperly or incompletely assembled. Use only original PARI parts and attachments authorized by PARI for the DURA-NEB 3000. Use of other parts and attachments may damage the unit or cause improper treatment. Do not open or disassemble the compressor unit (except for the Battery Access Door) at any time. Repairs, service and maintenance should be carried out only by an authorized PARI service center.

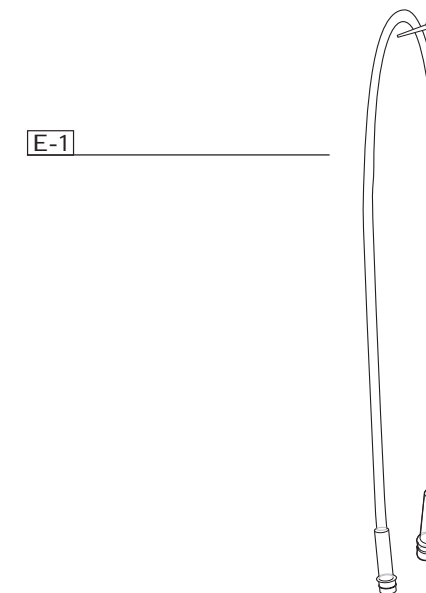
### CLEANING THE TUBING

If there is any moisture or condensation in the tubing, let the compressor run with only the tubing attached for 10 minutes. The warm air from the compressor will dry the moisture or condensation. Alternatively, you can dry the tubing by removing it from the compressor and hanging it up with the "ends down" [E-1] to allow any moisture or condensation to drain out and evaporate. Use a damp cloth to wipe the exterior of the tubing.

### FILTER REPLACEMENT

Under normal conditions, the filter on the front panel of the DURA-NEB 3000 should be replaced after 500 hours of use, or sooner if it appears soiled (grey), clogged or wet. Check the condition of the filter after every 10-15 treatments. Do not attempt to wash and re-use filters. Do not use substitute filter materials as compressor damage can result. Any attempt to use alternative filter materials will void compressor warranty and cause damage to the compressor. When replacing the filter, make sure that the front panel ON/OFF switch on the DURA-NEB 3000 is in the "OFF" position. Turn the round plastic filter cover counterclockwise (left) and remove filter (use a coin to help turn the cover if tight). The entire filter assembly was designed as a remove and replace item [E-2]. That means removing the filter, discarding it, and replacing it with a new filter assembly (part# 41F64).

Note: Do not attempt to disassemble filter assembly. Do not attempt to wash and re-use filter material. Do not attempt to replace the filter material only.

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## IX. Procedure For Service

- Contact PARI Patient Connection at 1.800.FAST.NEB (327.8632) or 804.639.7235 for shipping instructions. Press 2 for Patient Connection.
- In writing, describe the malfunction or symptoms of the DURA-NEB 3000. Include your full name, address and daytime telephone number.
- Send written description and DURA-NEB 3000 in a protected box to the address provided by Patient Connection.

## X. Limited Warranty

PARI offers a limited two (2) year warranty on the DURA-NEB 3000 compressor, and six (6) months on accessories and the internal battery. PARI warrants the DURA-NEB 3000 compressor and its parts will be free from defects in workmanship and materials for a period of two (2) years from date of first retail purchase of the compressor, and six (6) months on accessories and the internal battery. This warranty does not cover failures of the nebulizer or tubing. This warranty does not extend to failures resulting from accident, misuse, abuse, alteration, use of unauthorized service, parts or cleaning solutions, or failure to comply with

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Instructions For Use. If the warranted equipment should fail during the warranty period, PARI, at its option, will repair or replace the equipment or parts at issue, provided PARI determines to its satisfaction the claim is bona fide. It shall be the responsibility of the purchaser to pack and return the equipment in such a manner as to avoid shipping damage. The shipment cost must be pre-paid by customer to PARI Respiratory Equipment, Inc. This warranty covers the cost of labor incurred in the removal or replacement of the warranted component parts at authorized PARI service centers only. To make claim under this warranty, notify the authorized service center by calling PARI Patient Connection at:

1.800.FAST.NEB (327.8632), Tel 804.639.7235, Fax 804.639.7761

THERE ARE NO OTHER EXPRESS WARRANTIES. ALL IMPLIED WARRANTIES ON THE DURA-NEB 3000 AND ITS PARTS, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO TWO (2) YEARS FROM DATE OF FIRST RETAIL PURCHASE OF THE COMPRESSOR, AND SIX (6) MONTHS ON ACCESSORIES AND INTERNAL BATTERY. ALL EXPRESS OR IMPLIED WARRANTIES ON ANY OTHER PARTS, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE

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DISCLAIMED. Some states do not allow time limitations on implied warranties, so the above limitations may not apply to your situation. The sole and exclusive remedy under this warranty is to repair or replace the DURA-NEB 3000 and/or its parts at the option of PARI. IN NO EVENT SHALL PARI BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO THE PURCHASER OR ANY THIRD PARTY, INCLUDING WITHOUT LIMITATION, DAMAGES ARISING FROM PERSONAL INJURIES, LOST PROFITS, LOSS OF BUSINESS OPPORTUNITY, LOSS OF PROPERTY, ECONOMIC LOSSES, OR STATUTORY OR EXEMPLARY DAMAGES, WHETHER IN NEGLIGENCE, WARRANTY, STRICT LIABILITY OR OTHERWISE. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific rights. You may also have other legal rights, which vary, from state to state.

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## XI. Why You Should Complete And Mail Your Warranty Card

- It assures that your warranty is valid
- It will serve as proof of purchase for any insurance claims (fire, theft, flood).
- Your input is important for us to develop new products for other patients in the future.
- You will be entered in our quarterly "PARI Products Giveaway". Once completed, remove warranty card by folding and tearing along the perforations, and mail.

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**DATE OF PURCHASE:** \_\_\_\_\_  
**PURCHASED FROM:** \_\_\_\_\_  
 Home Healthcare Dealer  Pharmacy  Catalog Company  Other \_\_\_\_\_  
 Name of location where purchased: \_\_\_\_\_  
**COMPRESSOR SYSTEM:**  
 DURA-NEB®3000 WITH BATTERY  DURA-NEB®3000 WITHOUT BATTERY  
 Serial# of Compressor: \_\_\_\_\_  
 (LOOK FOR LABEL ON THE BACK OF THE UNIT)  
**REUSABLE NEBULIZER:**  
 PARI LC PLUS™  Other \_\_\_\_\_  
 Lot# of Nebulizer: \_\_\_\_\_  
 (A 6 DIGIT NUMBER IS INSCRIBED ON THE OUTSIDE OF THE NEBULIZER BODY BY THE PARI LOGO)  
**PATIENT'S PHYSICIAN SPECIALTY:**  
 Allergist  Internist  Family/General Practice  Pediatrician  
 Pulmonologist  Other \_\_\_\_\_  
 Physician Name: \_\_\_\_\_  
**INSURANCE COVERAGE:**  
 Medicare  Medicaid/Medical  Private Insurance (HMO, PPO, etc.)  
 Private Pay  Other \_\_\_\_\_ Patient Age: \_\_\_\_\_  
**PATIENT'S DIAGNOSIS:**  
 Asthma  Chronic Obstructive Pulmonary Disease (COPD)  
 Cystic Fibrosis  Other \_\_\_\_\_

CUSTOMER NAME \_\_\_\_\_  
 ADDRESS \_\_\_\_\_  
 CITY \_\_\_\_\_  
 STATE \_\_\_\_\_  
 ZIP \_\_\_\_\_  
 PHONE NO. (OPTIONAL) \_\_\_\_\_  
 E-MAIL \_\_\_\_\_

Send completed card to:  
**PARI RESPIRATORY EQUIPMENT, INC.**  
 13800 Hull Street Road, Midlothian, VA 23112-9939  
 1.800.FAST.NEB(327.8632), Teléfono 804.639.7235, Fax 804.639.7242

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Send completed card to:  
**PARI RESPIRATORY EQUIPMENT, INC.**  
 13800 Hull Street Road, Midlothian, VA 23112-9939  
 1.800.FAST.NEB(327.8632), Teléfono 804.639.7235, Fax 804.639.7242

DATE OF PURCHASE: \_\_\_\_\_

**PURCHASED FROM:**

Home Healthcare Dealer  Pharmacy  Catalog Company  Other \_\_\_\_\_

Name of location where purchased: \_\_\_\_\_

**COMPRESSOR SYSTEM:**

DURA-NEB®3000 WITH BATTERY  DURA-NEB®3000 WITHOUT BATTERY

Serial# of Compressor: \_\_\_\_\_  
(LOOK FOR LABEL ON THE BACK OF THE UNIT)

**REUSABLE NEBULIZER:**

PARI LC PLUS™  Other \_\_\_\_\_

Lot# of Nebulizer: \_\_\_\_\_  
(A 6 DIGIT NUMBER IS INSCRIBED ON THE OUTSIDE OF THE NEBULIZER BODY BY THE PARI LOGO)

**PATIENT'S PHYSICIAN SPECIALTY:**

Allergist  Internist  Family/General Practice  Pediatrician  
 Pulmonologist  Other \_\_\_\_\_

Physician Name: \_\_\_\_\_

**INSURANCE COVERAGE:**

Medicare  Medicaid/Medical  Private Insurance (HMO, PPO, etc.)  
 Private Pay  Other \_\_\_\_\_ Patient Age: \_\_\_\_\_

**PATIENT'S DIAGNOSIS:**

Asthma  Chronic Obstructive Pulmonary Disease (COPD)  
 Cystic Fibrosis  Other \_\_\_\_\_

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